



# Height Adjustable Medical Carts

## Operations & Maintenance Manual



## Symbols used in this manual:



Warning symbol. The warning symbol is used to identify critical hazards.



Important Note symbol. This symbol is used to identify important information that should be remembered.

## Table of Contents:

Symbols used in this manual	2
Description	4
Purpose	4
Features	5
Specifications	5
Moving the Height-Adjustable Medical Cart	6
Serial Number Location	7
Manufacturer Contact Information	7
Service Parts	7
Safety and Risks	8
Cleaning Instructions	8
Preventative Maintenance List	9
Warranty	10

## Description

- Designed for ease of cleaning and maintenance, with infection prevention of paramount importance, our **Height-Adjustable Medical Carts** are constructed of the highest quality materials and designed for long-lasting durability in solid colors or warm wood-grain finishes.
- The **Height-Adjustable Medical Cart** has two distinct parts, a height-adjustable work surface and a 3-drawer mobile storage cabinet.
- The **Height-Adjustable Medical Cart** can be manufactured with an optional detachable sharps holder and also an optional code lock on the top drawer.

## Purpose

- The functional design of the **Height-Adjustable Medical Cart** accommodates the storage and transport of care items for a patient in a hospital department.
- The **Height-Adjustable Medical Cart** is intended for use by professionally licensed staff.
- The **Height-Adjustable Medical Cart** has been designed and manufactured under ISO 13485 standards.
- The **Height-Adjustable Medical Cart** is intended for indoor use only.



**The improper use of the Height-Adjustable Medical Cart can lead to injury to the healthcare professional operating the Height-Adjustable Medical Cart. Use the Height-Adjustable Medical Cart only as intended, as defined in this manual.**



**Changes made to the Height-Adjustable Medical Cart or its parts can lead to injury to the healthcare professional operating the Height-Adjustable Medical Cart. Making any changes to the Height-Adjustable Medical Cart also voids its warranty.**

## Features:

Two design goals for the **Height-Adjustable Medical Cart** were lower cost of ownership for the hospital and improving the experience of the caregiver in the treatment environment.

1. KYDEX® thermofilm has been used on the secondary spill-containment work surface and the base panel for impact resistance and durability.
2. The Height-Adjustable Medical Cart has 5" casters –2 swivel, 1 locking and 1 steering/locking. The caster locks hold the Height-Adjustable Medical Cart firmly in place.
3. The base panel has been designed with concave curves to allow the caregiver to stand close to the work surface, minimizing the risk of caregivers hitting their shins while working with the cart.


---

## Specifications:

- **Surface Dimensions:** 35.5in wide x 19.5in deep (cm)
- **Surface height - Low:** 32.75in (76 cm)
- **Surface height - High:** 48.625in (76 cm)
- **Spill Containment Top Dimensions:** 22in wide x 14.75in deep (56cm x 37cm)
- **Base Dimensions:** 36.2in wide x 21.7in wide (cm)

## Operation: Moving the Height-Adjustable Medical Cart

1. Stand at either the left or right side of the Height-Adjustable Medical Cart.
2. Make certain the Height-Adjustable Medical Cart drawers are closed.
3. Make certain the casters brakes are in the off position.
4. Place both hands on the Height-Adjustable Medical Cart and push forward gently.
5. Make certain that the Height-Adjustable Medical Cart is always in front of you. Do not pull the Height-Adjustable Medical Cart behind you.

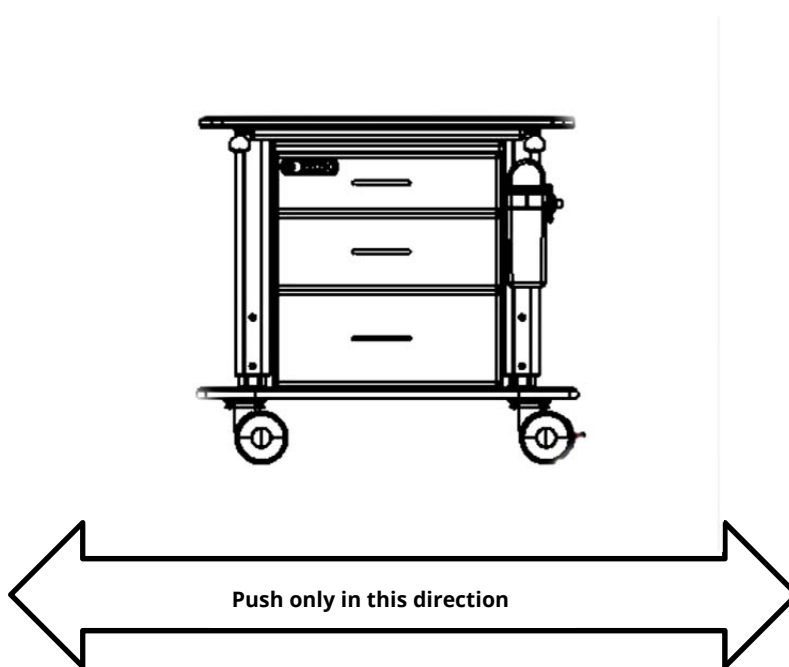

**Never push the Height-Adjustable Medical Cart from the front or rear.**



**Never pull the Height-Adjustable Medical Cart behind you.**

## Serial Number Location

The serial number is located on the exterior of the right side panel of the top drawer box.

---

## Contact Information

**Corporate Headquarters:**

MedViron LLC  
9380 Pentatech Drive  
Zeeland, Michigan 49464 USA

**Email:** [customerservice@medviron.com](mailto:customerservice@medviron.com)

**Toll free:** 1-844-633-8476

**Website:** [medviron.com](http://medviron.com)

**Mailing address:**

MedViron LLC  
P.O. Box 286  
Zeeland, Michigan 49464 USA

---

## Height-Adjustable Medical Cart Service Parts

Part Number	Part Name
557-0105-0000	Brushed Nickel bow pull
556-0073-0000	Caster Replacement Kit (4)

## Safety and Risks:



**The maximum weight load of each of the Height-Adjustable Medical Cart drawers is 10 pounds.**



**When moving the Height-Adjustable Medical Cart, the operator should push only from the either end.**



**To avoid tipping hazards, make certain that the all drawers and casters are secure when moving the Height-Adjustable Medical Cart.**

## Product Care and Disinfection

Clean with a damp cloth or mild detergent solution. MedViron furniture can withstand a wide range of hospital cleaners and disinfectants. When using any of these products, read and follow the manufactures directions carefully. Do not use abrasive or solvent cleaner. A CDC recommended chlorinated bleach solution of 5.25% -6.15% sodium hypochlorite (i.e., household bleach) diluted in water at 1:100 also can be used.

Always use a soft cloth or sponge on the Height-Adjustable Medical Cart for washing and rinsing. Always follow up the application of any cleaning solution with a warm water rinse.



**After cleaning, it is important that the Height-Adjustable Medical Cart is rinsed with warm water using a clean damp cloth and then wiped with a clean dry cloth to remove any moisture.**



### Preventative Maintenance List:

- \_\_\_ Check the casters. Casters should turn smoothly and should be free of any debris. Caster brakes should hold firmly, the wheel should not turn when brake is applied.
  
- \_\_\_ Check the drawer slides. Drawers should open smoothly and quietly.
  
- \_\_\_ Check labeling. Warning labels must be intact and legible.

Serial number	Date PM Completed	Completed by

## Limited Lifetime Warranty

**MedViron**<sup>®</sup> warrants the products it manufactures against defects in material and workmanship under normal use and service for the expected useful life of the product from the date of shipment. This limited warranty is valid only to the original purchaser using the product for its intended purpose. This warranty does not apply to expected changes resulting from normal wear and tear, use and aging of the product. Specifically excluded from this limited warranty are defects and damage resulting from abuse, neglect, misuse, or excessive use. Product alteration, substitution of components or materials, improper cleaning or repair, and installation and handling damages are also not covered by this warranty. All other express or implied warranties made by agents, dealers, sales representatives or any other representatives, including any warranty of merchantability or fitness for a particular purpose, are hereby excluded.

There are no representations as to the capacity or performance of the products sold hereunder except as set forth in the price list or quotation specifications, if any, and such representations are expressly conditioned upon the correctness of the data furnished by purchaser and upon the products being properly installed and maintained.

**MedViron's** sole obligation shall be to repair or replace any defective product or product component covered under this warranty, including shipping and labor, during the warranty period. In no event shall **MedViron** be liable for any consequential or incidental damages, including lost profits or revenues incurred by the purchaser with respect to any goods furnished by MedViron. Expenses incurred by purchaser in repairing or replacing MedViron product must be pre-approved in writing by **MedViron**

In the event a product has to be returned to the factory for verification of a defect and/or repair, it must be pre-approved in writing by **MedViron** freight pre-paid, and include a copy of the original invoice with the date and name of original purchaser.

**MedViron** hereby assigns to the original owner any and all warranties offered by its component suppliers. To the extent the terms of these warranties differ from the terms of the Limited Warranty, the terms of the suppliers' component warranties shall apply. However, **MedViron** does not guarantee that the component supplier will comply with any terms of the warranty of such supplier.

Any **MedViron** products with the serial number removed or altered are excluded from this warranty.

**MEDVIRON** WILL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES INCLUDING ANY LOSS, EXPENSE, OR DAMAGE OTHER THAN TO THE PRODUCT ITSELF THAT MAY RESULT FROM A DEFECT IN THE PRODUCT. SOME STATES DO NOT ALLOW FOR THE EXCLUSION OR LIMITATION OF CONSEQUENTIAL AND INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY.

**MEDVIRON** MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. SOME STATES DO NOT ALLOW AN EXCLUSION OR LIMITATION ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY.

THIS WARRANTY GIVES YOU, THE ORIGINAL PURCHASER, SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS THAT VARY FROM STATE TO STATE. EXCEPT FOR SUCH OTHER RIGHTS, THE REMEDIES PROVIDED UNDER THIS WARRANTY STATE THE LIMIT OF MEDVIRON'S RESPONSIBILITIES.