



# Rise Bassinets

## Operations & Maintenance Manual

**WARNING:** To avoid tipping hazards of this device, make sure that the basket or bed component sits firmly in the base and that all doors, drawers and casters are secure.



## Symbols used in this manual:



**Warning.** The warning symbol is used to identify critical hazards.



**Pinch point.** This symbol is used to identify a location where an injury may occur.



**Important Note.** This symbol is used to identify important information to remember.

## Contact

Corporate Headquarters:

Medviron LLC  
9380 Pentatech Drive  
Zeeland, Michigan 49464 USA

Email: [customerservice@medviron.com](mailto:customerservice@medviron.com)

Toll free: 1-844-633-8476

Website: [medviron.com](http://medviron.com)

Mailing address:

Medviron LLC  
P.O. Box 286  
Zeeland, Michigan 49464 USA

## Contents:

Symbols used in this manual	2
Bassinet Description	4
Purpose	4
Features	5
Specifications	6
Contraindications	6
Operating Instructions:	
Moving the Bassinet	7
Inclined Positioning	8
Serial Number Location	9
Manufacturer Contact Information	9
Safety and Risks	10
Cleaning Instructions	12
Preventative Maintenance List	13
Service Part List	14
Warranty	15

## Description

MedViron Rise Bassinets have two distinct parts, a plastic tub with mattress which sits on a mobile storage cabinet with drawers and easily adjustable surface height. The bassinet's modern design conveys the warmth of home and also implies competent, current healthcare.

## Purpose

The functional design of the bassinet accommodates the care and transport of an infant patient, the capability of inclined positioning of the bassinet tub, the storage of care items for the infant and height adjustability.

The bassinet is intended to be sold to and for use by professionally licensed staff and facilities caring for the labor and delivery of children.

The bassinet has been designed and manufactured under and to all pertaining ISO 13485 standards.

The bassinet is intended for indoor use only.



**THE IMPROPER USE OF THE BASSINET CAN LEAD TO INJURY TO THE INFANT OR TO THE HEALTHCARE PROFESSIONAL OPERATING THE BASSINET. USE THE BASSINET ONLY AS INTENDED, AS DEFINED IN THIS MANUAL.**



**CHANGES MADE TO THE BASSINET OR ITS PARTS CAN LEAD TO INJURY TO THE INFANT OR THE HEALTHCARE PROFESSIONAL OPERATING THE BASSINET. MAKING ANY CHANGES TO THE BASSINET ALSO VOIDS ITS WARRANTY.**

### Features:

Two design goals for the bassinet were lower cost of ownership for the hospital and improving the experience of mother, caregiver, and family.

1. Four height-adjustment handles, one in each corner, can be used independently to raise or lower the bassinet's height.
  2. One or two optional flip-up work surfaces are available upon request. The work surfaces have spill containment to capture spilled liquids.
  3. The bassinet's stainless steel push handles and all surface areas are easily accessible and easy to clean. The tub and handle design allow a clear view of the infant.
  4. The infant tub can easily be placed into an inclined position.
  5. The base panel has been designed to reduce the chances of caregivers hitting their shins while the bassinet is in motion. The base panel's concave curves also allow caregivers to stand closer while caring for the infant.
  6. KYDEX<sup>®</sup> thermoplastic has been used for cleanability, impact resistance and durability.
-

## Specifications:



Width	37.5in (95 cm)
Depth	21.5in (55 cm)
Surface low height	29.5in (75 cm)
Surface high height	39in (99 cm)
Drawer dimensions	23in W x 15in L x 6in D

## Contraindications:

1. The bassinet tub is not intended to be used without the mobile storage cabinet beneath.
2. The bassinet's plastic drawer is not intended to be used for anything other than as a drawer.
3. The bassinet is not intended to be used for infants that can sit, stand, or roll over.
4. The bassinet tub or drawer boxes are not intended to be used for bathing.

## Operation: Moving the Bassinet

1. Stand at either the left or right side of the bassinet.
2. Make certain the bassinet tub is secure and that drawers are closed. Flip-up Work Surfaces should be in the down position as shown below.
3. Place both hands on the bassinet push handle and push forward gently.
4. Make certain that the bassinet is always in front of you. Do not pull the bassinet behind you.
5. Make certain that the floor is free of obstruction and beware of transitions in the floor.



**NEVER PUSH THE BASSINET FROM THE FRONT OR REAR.**



**NEVER PULL THE BASSINET BEHIND YOU.**




**TRANSPORT OF AN INFANT MUST OCCUR IN THE LOWEST HEIGHT POSITION ONLY.**

## Operation: Inclined Infant Tub Positioning

To put the bassinet tub into an inclined position, simply raise the tallest end of the tub and then lower it so that the hook on the side of the tub engages the metal rail that bisects the push handle.

Lift this side and engage the hook for the inclined position.

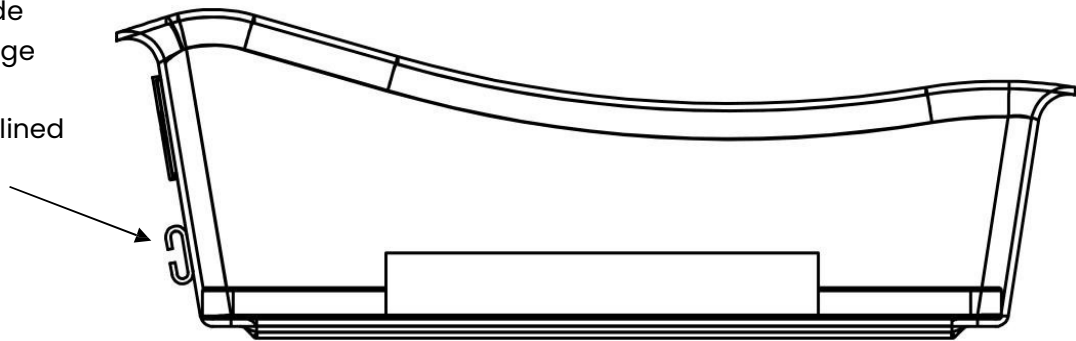

**DO NOT LET GO OF THE TUB UNTIL YOU ARE CERTAIN THE TUB HOOK IS FIRMLY SEATED ON THE RAIL.**



**NEVER PLACE ANY FOREIGN OBJECTS INSIDE OR ON TOP OF THE BASSINET TUB.**



## Serial Number & Cleaning Instructions Location



Serial number label is on the INSIDE of the side panel.



The serial number and cleaning instructions are on a label on the INSIDE of the side panel. Stand in front of the bassinet and push the lower pass-thru drawer inward, away from you, to reveal the label.

## Contact Information

### Corporate Headquarters:

Medvion LLC  
9380 Pentatech Drive  
Zeeland, Michigan  
49464 USA

**Email:** [customerservice@medvion.com](mailto:customerservice@medvion.com)

### Mailing address:

Medvion LLC  
PO Box 286  
Zeeland, Michigan 49464 USA

**Toll free:** 1-844-633-8476

**Website:** [medvion.com](http://medvion.com)

## Safety and Risks:



**TO AVOID TIPPING HAZARDS OF THIS DEVICE, MAKE SURE THAT THE BASKET OR BED COMPONENT FITS FIRMLY IN THE BASE, AND THAT THE DRAWER AND CASTERS ARE SECURE.**



**TRANSPORT OF AN INFANT SHOULD OCCUR IN THE LOWEST HEIGHT POSITION ONLY.**



**WHEN PUTTING THE BASSINET TUB INTO THE INCLINED POSITION, ALWAYS VERIFY THAT THE TUB HOOK IS SECURE AND SEATED FIRMLY ON THE RAIL.**



**OBJECTS SHOULD NOT BE LEFT IN OR ON TOP OF THE BASSINET INFANT TUB AT ANY TIME.**



**THE MAXIMUM WEIGHT LOAD OF EACH OF THE BASSINET DRAWERS IS 15 POUNDS (7 KG). OPEN ONLY ONE DRAWER AT A TIME ON THE SAME SIDE. THIS IS NECESSARY TO AVOID TIPPING OF THE BASSINET WHEN THE DRAWERS ARE OPEN.**



**THE MAXIMUM WEIGHT LOAD OF THE FLIP-UP WORK SURFACE IS 10LBS (4.5 KG). THIS IS NECESSARY TO AVOID TIPPING THE BASSINET.**



**NEVER PLACE AN INFANT ON THE FLIP-UP WORK SURFACE.**



**WHEN MOVING THE BASSINET, THE OPERATOR SHOULD PUSH ONLY FROM THE EITHER END USING THE BASSINET HANDLES.**



**TO AVOID TIPPING HAZARDS OF THIS DEVICE, MAKE CERTAIN THAT THE BED COMPONENT FITS FIRMLY IN THE BASE, AND THAT DRAWERS AND CASTERS ARE SECURE.**



**PINCH POINTS EXIST AT THE LOCATIONS INDICATED IN THE DRAWINGS BELOW. PLEASE BE CAREFUL.**

Pinch points



Pinch point

## Product Care and Disinfection

Clean with a damp cloth or mild detergent solution. MedViron products can withstand a wide range of hospital cleaners and disinfectants. When using any of these products, read and follow the manufacturer's directions carefully. Do not use abrasive or solvent cleaner. A CDC recommended chlorinated bleach solution of 5.25% -6.15% sodium hypochlorite (i.e., household bleach) diluted in water at 1:100 also can be used.

**Infant tub:** Always use a soft cloth or sponge on the bassinet's infant tub for washing and rinsing. Liquid dish detergent is an acceptable cleaner. Never use abrasives or solvent cleaners. Always follow up the application of any cleaning solution with a warm water rinse.

Clean the tub and mattress after each patient. Do not immerse the mattress in cleaning solution. Make certain the mattress and tub are completely dry after each cleaning.



After cleaning, it is important that the bassinet is rinsed with warm water using a clean damp cloth and then wiped with a clean dry cloth to remove any moisture.

## Preventative Maintenance List



**WARNING:** To avoid tipping hazards of this device, make sure that the basket or bed component sits firmly in the base and that all doors, drawers and casters are secure.

\_\_\_\_\_ Casters are secure. (Check that hex nuts are tight: 4 per caster)

\_\_\_\_\_ Casters: Casters should turn smoothly and should be free of any debris.  
Caster brakes should hold firmly, the wheel should not turn when brake is applied.

\_\_\_\_\_ Plastic drawer: Should be clean and free of damage.

\_\_\_\_\_ Drawer slides: Drawers should open smoothly and quietly.

\_\_\_\_\_ Bassinet tub: Incline hook must be intact and firmly attached to the tub. Bassinet tub should be free of damage.

\_\_\_\_\_ Warning labels and serial number labels must be intact and legible.

\_\_\_\_\_ Check mattress for tears, punctures, or wear.  
Verify that the flip-up work surface moves smoothly, and that the Flip-up Work Surface Support locks into position when raised.

Serial number	Date Completed	Completed by

## Service Parts

Part Name	Part Number
Bassinet Tub	523-0150-0000
Mattress	516-0000-0000
Inset Drawer Pull	557-0105-0000
Stainless Steel Drawer Pull	557-0036-0028
Plastic Drawer Liner	558-0063-0000
Caster Replacement Kit (4 casters w/hardware)	556-0305-0000
Flip-up Work Surface Support	555-0097-0000
Flip-up Work Surface Hinge	555-0055-0000
Flip-up Work Surface (color specific)	510-M010-Kxxx
Drawer Fronts (color specific)	530-M009-Kxxx



Please contact your dealer or sales representative to order service parts.

## Limited Lifetime Warranty

MedViron® warrants the products it manufactures against defects in material and workmanship under normal use and service for the expected useful life of the product from the date of shipment. This limited warranty is valid only to the original purchaser using the product for its intended purpose. This warranty does not apply to the expected changes resulting from normal wear and tear, use and aging of the product. Specifically excluded from this limited warranty are defects and damage resulting from abuse, neglect, misuse, or excessive use. Product alteration, substitution of components or materials, improper cleaning or repair, and installation and handling damages are also not covered by this warranty. All other express or implied warranties made by agents, dealers, sales representatives, or any other representatives, including any warranty of merchantability or fitness for a particular purpose, are hereby excluded.

There are no representations as to the capacity or performance of the products sold hereunder except as set forth in the price list or quotation specifications, if any, and such representations are expressly conditioned upon the correctness of the data furnished by purchaser and upon the products being properly installed and maintained.

MedViron's sole obligation shall be to repair or replace any defective product or product component covered under this warranty, including shipping and labor, during the warranty period. In no event shall MedViron be liable for any consequential or incidental damages, including lost profits or revenues incurred by the purchaser with respect to any goods furnished by MedViron. Expenses incurred by purchaser in repairing or replacing MedViron product must be pre-approved in writing by MedViron.

In the event a product has to be returned to the factory for verification of a defect and/or repair, it must be pre-approved in writing by MedViron freight pre-paid and include a copy of the original invoice with the date and name of the original purchaser.

MedViron hereby assigns to the original owner any and all warranties offered by its component suppliers. To the extent the terms of these warranties differ from the terms of the Limited Warranty, the terms of the suppliers' component warranties shall apply. However, MedViron does not guarantee that the component supplier will comply with any terms of the warranty of such supplier.

Any MedViron products with the serial number removed or altered are excluded from this warranty.

MEDVIRON WILL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES INCLUDING ANY LOSS, EXPENSE, OR DAMAGE OTHER THAN TO THE PRODUCT ITSELF THAT MAY RESULT FROM A DEFECT IN THE PRODUCT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF CONSEQUENTIAL AND INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY.

MEDVIRON MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. SOME STATES DO NOT ALLOW AN EXCLUSION OR LIMITATION ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY.

THIS WARRANTY GIVES YOU, THE ORIGINAL PURCHASER, SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS THAT VARY FROM STATE TO STATE. EXCEPT FOR SUCH OTHER RIGHTS, THE REMEDIES PROVIDED UNDER THIS WARRANTY STATE THE LIMIT OF MEDVIRON'S RESPONSIBILITIES.

## Notes