



Mobile Laptop Carts

User Manual



Contents

Thank you	3
<hr/>	
Safety Alerts	3
<hr/>	
Warnings	4
<hr/>	
Raising and Lowering the Top Surface	5
<hr/>	
Cleaning and Disinfection	6
<hr/>	
Specifications	6
<hr/>	
Preventative Maintenance	7
<hr/>	
Serial Number Location	8
<hr/>	
Manufacturer Contact Information	8
<hr/>	

Thank you for purchasing a medViron Mobile Laptop Cart.

The medViron Mobile Laptop Cart is a mobile work surface with an adjustable height surface.

Please become familiar with all details in this manual before using the Mobile Laptop Cart or beginning maintenance. Procedures should be established for training staff on the proper use and operation of your medViron Mobile Laptop Cart.

Safety Alerts:

- This user guide must be read completely before the Mobile Laptop Cart is put into use.
- The Mobile Laptop Cart should only be used on flat surfaces.
- The casters are designed for use in an indoor environment and for travelling on flat, smooth, clean floors.
- The casters could be damaged when moving the Mobile Laptop Cart over rough, uneven, and dirty surfaces.
- Do not use the Mobile Laptop Cart if its construction or casters have been damaged because this could cause loss of stability.
- Do not sit or lean on the Mobile Laptop Cart. Doing so may cause the cart to tip which may result in personal injury or damage to the cart.
- Do not use the Mobile Laptop Cart as a support when getting up or as an assist in walking.
- Do not overload the top surface more than the prescribed limits.
- Exercise caution when operating the activation handle to avoid pinching fingers between the activation handle and top surface.
- If defects are found on any part of the Mobile Laptop Cart, immediately remove the Mobile Laptop Cart from service.

- Only original materials and components may be used to repair the Mobile Laptop Cart. MedViron will not be liable for any damages that arise from using non-original or inappropriate materials and components.
- Follow cleaning and preventative maintenance instructions.

Warnings:



Do not lift the Mobile Laptop Cart by the top when it is in the full extension position.



Do not attempt in any manner to alter, adjust, or lubricate the internal contents of the column, including gas spring and mechanisms. Doing so will void the warranty and may cause permanent damage to the Mobile Laptop Cart.




Loading the Mobile Laptop Cart beyond the safe working load could result in injury, or cause damage to the Mobile Laptop Cart.

Raising the Mobile Laptop Cart

To raise the Mobile Laptop Cart, lift the height-adjustment activation handle upward. The surface will rise gently. If there is weight on the top surface, you may need to lift the work surface gently to get it to rise.

Lowering the Mobile Laptop Cart

First engage the activation handle by lifting it upward, then apply downward pressure on the top surface.

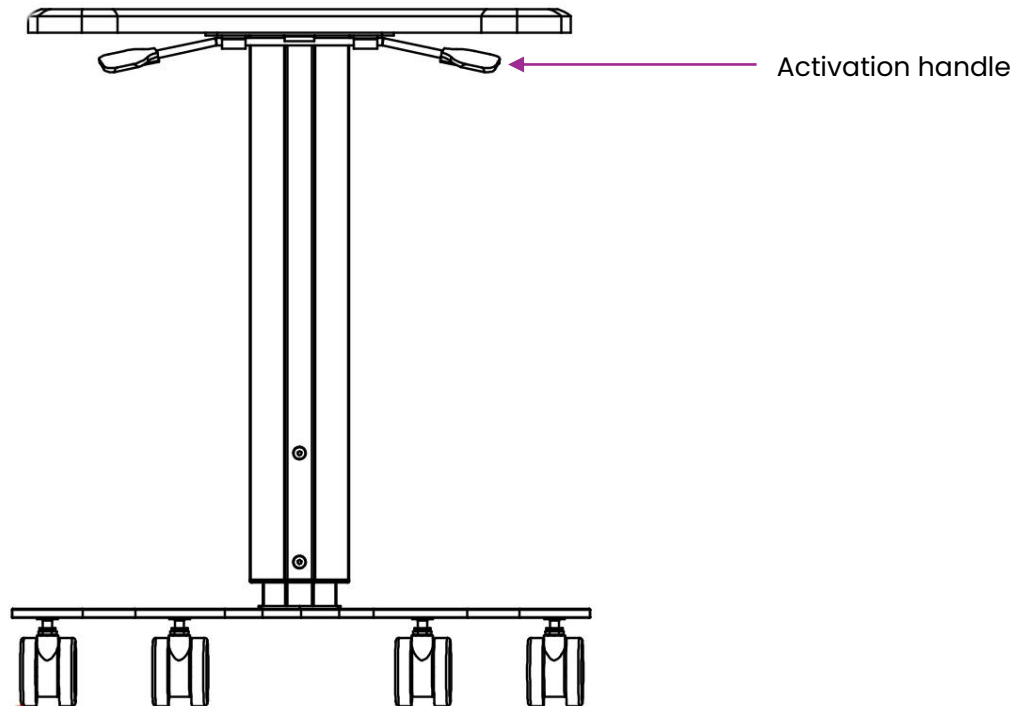

Cleaning and Disinfection

The Centers for Disease Control (CDC) recommends that surfaces should be cleaned routinely and when dirty or soiled to prevent potentially contaminated objects from serving as a source of healthcare associated infections. MedViron Mobile Laptop Carts can withstand a wide range of hospital cleaners and disinfectants. Follow the cleaning solution manufacturer's instructions when using these products. The CDC recommends using a 5.25% - 6.15% sodium hypochlorite chlorinated bleach solution (i.e., household bleach) diluted 1:100 with water. Be sure to thoroughly rinse the solution from the surface and remove any moisture from the surface with a dry cloth.¹

Mobile Laptop Cart Specifications

Specifications	
Safe Working Load	25lbs (11.4 kg)
Surface Width	23" (58 cm)
Surface Depth	17" (43 cm)
Surface Height	30" to 46" (76 cm to 1.17 m)
Base Width	24.5" wide (62 cm)
Base Depth	17" deep (43 cm)
Base Height	4" (10 cm)
Weight	42lbs (19 kg)
Warranty	Limited Lifetime Warranty

¹ The Centers of Disease Control. Guideline for Disinfection and Sterilization in Healthcare Facilities, 2008 by William A. Rutala, Ph.D., M.P.H., David J. Weber, M.D., M.P.H., and the Healthcare Infection Control Advisory Committee, Nov. 2008.

Preventative Maintenance

Preventative maintenance is recommended at a minimum of once per year.

Checklist:

_____ All fasteners are secure.

_____ All casters are secure, free from debris, and roll and swivel freely.

_____ The height-adjustment activation handle operates properly.

_____ Top surface intact and undamaged.


Maintenance Log

Serial Number	Date Completed	Completed by

Serial Number Location

Your Mobile Laptop Cart serial number is required for all written and verbal communication with your dealer or medViron customer service. The serial number is located on a label that is on the steel plate that supports the Mobile Laptop Cart top surface. The serial number is located directly above the bar code.

Bottom view:

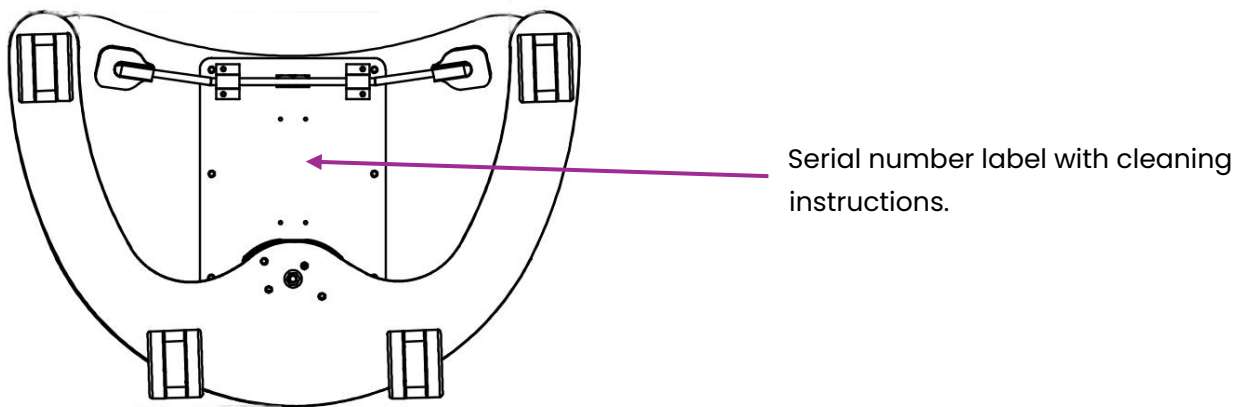

Manufacturer's Contact Information

Corporate Headquarters:

medViron LLC
9380 Pentatech Drive
Zeeland, Michigan 49464 USA

Website: medviron.com

Customer Service Phone: 844-633-8476

Customer Service Email: customerservice@medviron.com