



Roam 2.0 & Roam 2.0XL

User Manual



Contents

Thank you	3
<hr/>	
Safety Alerts	3
<hr/>	
Warnings	4
<hr/>	
Raising and Lowering the Top Surface	5
<hr/>	
Cleaning and Disinfection	6
<hr/>	
Specifications	6
<hr/>	
Preventative Maintenance	7
<hr/>	
Serial Number Location	8
<hr/>	
Manufacturer Contact Information	8
<hr/>	

Thank you for purchasing a medViron Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface.

MedViron's Roam 2.0 and Roam 2.0XL are mobile work surfaces with an adjustable height surface.

Please become familiar with all details in this manual before using Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface or beginning maintenance. Procedures should be established for training staff in the proper use and operation of your Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface.

Safety Alerts:

- This user guide must be read completely before Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface is put into use.
- The Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface should only be used on flat surfaces.
- The casters are designed for use in an indoor environment and for travelling on flat, smooth, clean floors.
- The casters could be damaged when moving Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface over rough, uneven, and dirty surfaces.
- Do not use the Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface if its construction or casters have been damaged because this could cause loss of stability.
- Do not sit or lean on the Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface. Doing so may cause it to tip which may result in personal injury or damage.
- Do not use Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface as support when getting up or as an assist in walking.
- Do not overload the top surface more than the prescribed limits.

- Exercise caution when operating the activation handle to avoid pinching fingers between the activation handle and top surface.
- If defects are found on any part of the Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface, immediately remove it from service.
- Only original materials and components may be used to repair the Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface. MedViron will not be liable for any damages that arise from using non-original or inappropriate materials and components.
- Follow cleaning and preventative maintenance instructions.

Warnings:



Do not lift the Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface by the top when it is in full extension position.



Do not attempt in any manner to alter, adjust, or lubricate the internal contents of the column, including gas spring and mechanisms. Doing so will void the warranty and may cause permanent damage to the Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface.



Loading the Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface beyond the safe working load could result in injury, or cause damage to it.

Raising the Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface

To raise the Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface, LIFT the height-adjustment activation handle UPWARD. The surface will rise gently. If there is weight on the top surface, you may need to lift the work surface gently to get it to rise.

Lowering the Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface

First engage the height-adjustment activation handle by LIFTING it UPWARD, then apply downward pressure on the top surface.



Cleaning and Disinfection

The Centers for Disease Control (CDC) recommends that surfaces should be cleaned routinely and when dirty or soiled to prevent potentially contaminated objects from serving as a source of healthcare associated infections. MedViron Roam 2.0 or Roam 2.0XL Height Adjustable Work Surfaces can withstand a wide range of hospital cleaners and disinfectants. Follow the cleaning solution manufacturer's instructions when using these products. The CDC recommends using a 5.25% - 6.15% sodium hypochlorite chlorinated bleach solution (i.e., household bleach) diluted 1:100 with water. Be sure to thoroughly rinse the solution from the surface and remove any moisture from the surface with a dry cloth.¹

Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface Specifications

Specifications	Roam 2.0	Roam 2.0XL
Safe Working Load		
Surface Width		
Surface Depth		
Surface Height		
Base Width		
Base Depth		
Base Height		
Weight		
Warranty	Limited Lifetime Warranty	Limited Lifetime Warranty

¹ The Centers of Disease Control. Guideline for Disinfection and Sterilization in Healthcare Facilities, 2008 by William A. Rutala, Ph.D., M.P.H., David J. Weber, M.D., M.P.H., and the Healthcare Infection Control Advisory Committee, Nov. 2008.

Preventative Maintenance

Preventative maintenance is recommended at a minimum of once per year.

Checklist:

_____ All fasteners are secure.

_____ All casters are secure, free from debris, and roll and swivel freely.

_____ The height-adjustment activation handle operates properly.

_____ Top surface intact and undamaged.

Maintenance Log

Serial Number	Date Completed	Completed by

Serial Number Location

Your Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface serial number is required for all written and verbal communication with your dealer or medViron customer service. The serial number is located on a label that is on the steel plate that supports Roam 2.0 or Roam 2.0XL Height Adjustable Work Surface's top surface. The serial number is located directly above the bar code.

Manufacturer's Contact Information

Corporate Headquarters:

medViron LLC
9380 Pentatech Drive
Zeeland, Michigan 49464 USA

Website: medviron.com

Customer Service Phone: 844-633-8476

Customer Service Email: customerservice@medviron.com